

## Instruction for entering recalibration codes on InnovaStar with touch-screen

Recalibration codes oneHbA1 IS Lot xx:

First recalibration code: xxxxxxxx

Second recalibration code: xxxxxxxx

Third recalibration code: xxxxxxxx


Fourth recalibration code: xxxxxxxx

Fifth recalibration code: xxxxxxxx


Security code: xxx


All given recalibration codes and the security code have to be entered into the device as described below.


1. Switch on the device.
2. After the startup, the screen below will be shown. Press in the middle of the touch screen (marked red) to access the function screen.

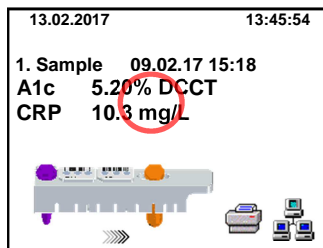

3. Touch the  icon to start recalibration.

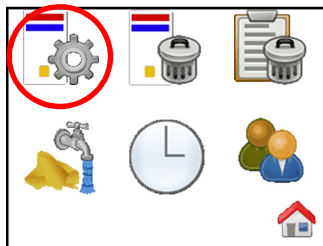



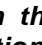
4. If different parameters/lots are stored, the  icon is used to scroll to the lot to be recalibrated.


5. If the desired lot is shown, please press the  icon to start the recalibration process.




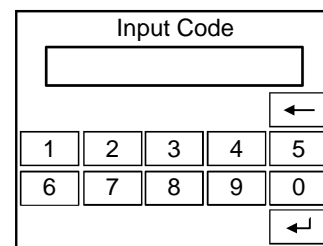
6. Please type in the first recalibration code using the virtual keypad.

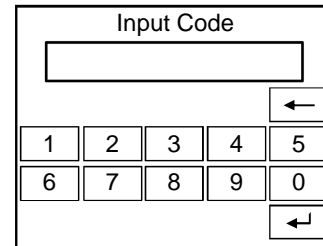


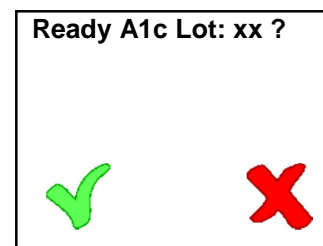
7. After confirmation of the first recalibration code by touching the  icon the following screen will appear. **Touch the  icon if more than one recalibration code has to be entered, otherwise proceed with step 9.**

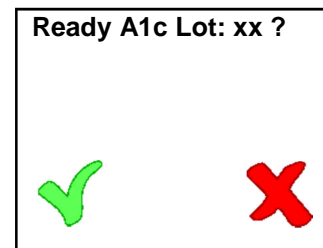




8. After pressing the  icon, the following screen will appear. Please enter the second/third/etc. recalibration code as described in step 6-7.




9. After entering **all** given recalibration codes, the following screen has to be confirmed by touching the  icon.



10. Enter the security code xxx and confirm by touching the  icon.

Security Code					
<input type="text"/>					
1	2	3	4	5	
6	7	8	9	0	
					

11. The selected HbA1c lot is recalibrated successfully when the screen below is shown.

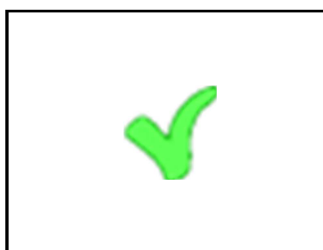

**Please note:**

This instruction for recalibration is also applicable for other parameters.